



IMPORTANT PRODUCT INFORMATION

Liquid Unassayed MultiquaI - Level 3

Important information on Liquid Unassayed MultiquaI - Level 3, Lot 56743

For the attention of: Clinical Laboratory Director, Manager or Supervisor /Laboratory Professional Users /Channel Partner

Please retain this letter for your records

Date: January 10, 2025

Bio-Rad Reference: FAR 2024-04B

Dear Valued Customer / Channel Partner,

The purpose of this letter is to inform you that you may observe lower calcium recovery in Level 3 of Liquid Unassayed MultiquaI, Lot 56743, when the product is initially thawed.

Reason for the Action:

Lower calcium recovery is observed when the product is initially thawed, followed with recovery increasing to the expected values within several hours after thaw.

Affected Product Identification:

Product Name		Liquid Unassayed MultiquaI, Level 3		
Level	Catalog Number	Lot Number	Lot Number Expiration Date	UDI NUMBER
3	699	56743	2026-08-31	00847661001324
MiniPak	698X	56740		00847661001935

Action(s) to be taken by the Customer:

For optimal performance and to minimize the impact of the initial low calcium recovery, the product requires an additional hold time of at least 24-48 hours in the refrigerator (2-8 °C) after the product is thawed and unopened, prior to use.

Bio-Rad is requesting that customers affected by this notice take the following action:

- Hold Lot 56743 (Level 3) for at least 24-48 hours in the refrigerator (2-8 °C) after the product is thawed and unopened prior to use.
- Ensure this notice is provided to all those who need to be aware within your organization or to any organization where the impacted devices have been transferred.
- Please complete and return the attached response form as soon as possible so that we are assured you have received this important communication.



**Bio-Rad
Laboratories**

*Clinical Diagnostics Group
9500 Jeronimo Road
Irvine, California 92618-2017
Tel: (949) 598-1200
www.bio-rad.com/qualitycontrol*

Resolution by Bio-Rad:

We are working to resolve the issue and apologize for any inconvenience.

Contact Information:

Please contact your local Bio-Rad Technical Support if you have any questions regarding this communication. USA contact information is provided below.

- Technical Support Department (USA) at 1-800-854-6737, option #2, or by email at qsd.techservice@bio-rad.com if you have technical questions regarding this notice.

Bio-Rad is committed to ensuring the highest levels of safety and quality. We sincerely apologize for any inconvenience this issue may have caused

Mario Wijker
Bio-Rad SVP, RAQA



FIELD ACTION RESPONSE FORM

Bio-Rad Reference: FAR 2024-04B

Bio-Rad Product Segment: Bio-Rad Laboratories Quality Systems (QS)

Single Registration Number (SRN): US-MF-000020712

Product Name		Liquid Unassayed Multiquel, Level 3		
Level	Catalog Number	Lot Number	Lot Number Expiration Date	UDI NUMBER
3	699	56743	2026-08-31	00847661001324
MiniPak	698X	56740		00847661001935

CUSTOMER / CHANNEL PARTNER INFORMATION

Account Name:	
Undersigning Manager Name:	
Address:	
Telephone Number / Fax:	
Account Number:	

STATEMENT:

- No affected product received
- I am aware of the information about the field action concerning the above reference product(s) and have proceeded according to the instructions issued by Bio-Rad.
- For completion by Channel Partners: All customers have been informed about this field action and have proceeded according to the instructions issued by Bio-Rad. Number of customers informed: _____

Number of affected products received:		Number of affected products corrected/ destroyed/ returned (as applicable to the Field Action instructions):	
If number of products corrected/ destroyed/ returned is different to the number received, please account for the difference:			

Date: _____

Customer / Channel Partner Signature (and Stamp if applicable):

Please return this form to: qsd_far@bio-rad.com or send it via fax to 949-598-1557 (Attention: Regulatory Affairs). We need this information to ensure that you have received the product notification.